Careva Pelvic KIT installation manual •  
Anvisning för montering av Careva Höft-KIT •  
Monteringsanvisning for Careva Hofte-KIT •  
Installationsanleitung für Careva Hüftset •  
Instructions de montage du kit ceinture de  
hanche Careva • Instrucciones para montar el  
Kit para la cadera Careva • Manual de  
instalação do Conjunto Pélvico •  
Инструкция по креплению набора Careva с  
набедренным ремнём

Careva AB  
Theres Svenssons gata 10  
S 41755 Gothenburg  
Sweden  
+46 (0) 31 222800  
[info@careva.se](mailto:info@careva.se)  
[www.careva.se](http://www.careva.se)

## EN

### Careva Pelvic KIT installation manual

The lap belt is used to prevent the person sliding forwards or underneath. There is the option to lock either at the front, or at the side. Always fit the spinal belt to the seat as per the manual before fitting the combi harness.

#### Front locking:

Item no. 10022 (S), 10021 (M), 10031 (L)

**1.** Loosen the harness strap from the padded section. Fix the harness strap in the channel on the spinal belt which is at the bottom of the seat and pull it halfway through the channel.

**2.** Place the remainder of the T-harness with the reverse side facing the seat. Open the velcro fastening and fix it in the same channel as the long strap.

The crotch strap is now fixed to the channel of the spinal belt and can be adjusted in length by using the velcro fastenings on the crotch section.

**3.** Place the person on top of the crotch strap on the chair and open the padded section over the pelvis. Feed the lock of the long strap through the elastic spring on each side of the padding and lock at the front. Tighten. The T-harness can also be locked at the back of the chair to prevent very active children and adults from undoing the harness. This option requires a longer harness strap and lock (item no. 10150).

#### Side locking:

T-harness item no. 10222 (S), 10221 (M), and 10231 (L) is locked at the side using a separate G-lock. The G-lock is fixed in the channel of the spinal belt (at the bottom of the seat). The harness is locked and adjusted afterwards using the straps on each side.

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